



BLOC Nurse Call

Exordium Support Responsibilities

What we can do

- Access Web portal
- See active alerts
- See Access point status
- Access reports
- Add users
- Manage what alerts go through to mobile app

Exordium
Networks Inc

Expectations

- Smooth functioning of nurse call system
- Ability to guide end-users to properly use the system
- Answer any questions management or care staff may have
- Awareness of features available on the system

EXORDIUM
Networks Inc

Web portal – configures and manages system instance on the cloud

Server Console – on-premises server for all transmitters and devices

Wireless coordinator and access points – receive data from transmitters and check them in

Transmitters – collect and transmit data like resident pulls, incontinence, etc.


Care staff – pick up and clear alerts using buttons on device or pendant reset process

Application/Console -- displays alerts and provides interface to clear/assign them

Components of the System

Web interface

Located at manage.bloctech.com

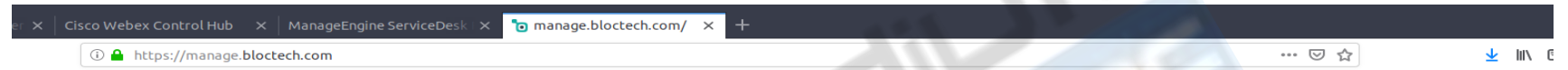

<https://manage.bloctech.com>



Login

Please log in with your Bloc credentials.

Username

Password

Log In

Configuration

Management

Reporting

Dealer Dashboard

System Setup

Three main functions



Dealer Settings

Save time by adding dealer information to automatically populate new system configurations.



Register a Bloc-SC

Register Bloc-SCs for new system setups or to replace a Bloc-SC on an existing system.



Start a New Job

After registering a Bloc-SC, start building a new system.



Select a System

Select a built system to manage it.

Two for installation

One for support

Systems Overview



Online Systems

2



Offline Systems

1



Unassigned Bloc-SCs

0

System Management



Select a System

Select a built system to manage it.


Select a System

Available Systems

Search...

Status	Community Name	Property ID	MAC Address	Upload Status
●	PSL Parmer Woods Austin	40016	00:80:A3:CB:AF:F4	Complete
●	PSO Bridgewood Gardens Albertville, AL	30011	00:80:A3:B9:FE:07	Complete
●	Highland Village Mansfield, TX	30014	00:80:A3:D1:35:C1	Complete

Select system. Red means offline

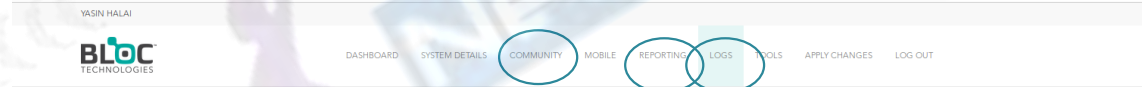

System Status Events

Current Event Status

There are currently no active alarms

Community – to manage apartments, zones, groups
Reporting – to see overall and filtered reports
Logs – Event and System logs show all active alerts

Apartments and Zones

In Community tab

YASIN HALAI

BLOC
TECHNOLOGIES

DASHBOARD SYSTEM DETAILS COMMUNITY MOBILE REPORTING LOGS TOOLS APPLY CHANGES LOG OUT

Zones

Apartment 10 Apartment 101 Apartment 102 Apartment 103 Apartment 104 Apartment 105 Apartment 106 Apartment 107

Apartment	Group	Zone Type	Check-In	Actions
Apartment 10	Memory Care	Wireless Apartment	No Check In	
Apartment 101	AL Apartments	Wireless Apartment	Standard	
Apartment 102	AL Apartments	Wireless Apartment	Standard	
Apartment 103	AL Apartments	Wireless Apartment	Standard	
Apartment 104	AL Apartments	Wireless Apartment	Standard	
Apartment 105	AL Apartments	Wireless Apartment	Standard	
Apartment 106	AL Apartments	Wireless Apartment	Standard	
Apartment 107	AL Apartments	Wireless Apartment	Standard	

Click to add text

Edit Zone

Long Name: Apartment 104

Short Name: Apt 104

Group: AL Apartments
Zone Type: Wireless Apartment, Check-In: Standard

Name A:

Name B:

Phone:

Type	Code	Supervised
Pendant Alarm A	7290	<input type="checkbox"/>
Smoke Warning	A2A6	<input checked="" type="checkbox"/>
Bath E-Call	D070	<input checked="" type="checkbox"/>
Emergency Call	D0D5	<input checked="" type="checkbox"/>